ACCUCHEK® Guide Me

Roche

EN

START HERE

QUICK START GUIDE

To find diabetes management apps that are compatible with your meter, visit www.accu-chek.com.

ACCUCHEK®

WARNING

Risk of life-threatening injuries
Keep new and used batteries away from children. See the User's Manual for additional information.

Risk of a serious health incident
This Quick Start Guide does not replace the User's Manual for your Accu-Chek Guide Me blood glucose meter. The User's Manual contains important handling instructions and additional information.

Risk of infection
While obtaining a blood drop, the lancing device and cap may come into contact with blood. The lancing device is intended for personal use only.

NOTE

- Always have a spare set of batteries.
- Battery life may vary due to factors such as temperature and battery manufacturer.

Overview

- 1 Power/Set Button
- 2 Display
- 3 Left Arrow Button
- 4 Right Arrow Button
- 5 Test Strip Slot
- 6 Battery Door
- 7 Micro USB Port

Set the Time and Date

1

The meter prompts you to set the time and date the first time you turn it on. **Set-up** appears on the display with the hour flashing.

Press or to decrease or increase the hour.

2

Press to set the hour and move to the next field.

Repeat steps 1 and 2 to adjust each field. Press and hold until the flashing test strip symbol appears.

Set Up the Lancing Device

1

Familiarize yourself with the Accu-Chek Softclix lancing device.

2

Remove the lancing device cap by pulling it straight off. Do not twist the cap.

3

Insert a lancet into the lancing device until it clicks.

4

Twist off the lancet's protective cap.

5

Place the cap back on the lancing device. Make sure the notch on the cap lines up with the notch on the lancing device.

6

Adjust the lancet depth to a comfortable penetration depth. Start at 2. For tougher skin, rotate to a higher number.

7

Press the priming button in as far as it will go, like a pen. Do not press the release button while pressing the priming button.

8

The release button turns yellow when the lancing device is ready. Set the lancing device aside until you are ready to perform a fingerstick.

Perform a Blood Glucose Test

1

Wash and dry your hands before you perform a blood glucose test.

2

Check the use by date on the test strip container. Do not use test strips past the use by date.

3

Remove a test strip from the test strip container. Close the cap tightly.

4

Insert the metallic end of the test strip into the meter.

The meter turns on. A flashing drop symbol appears.

5

Press the lancing device firmly against the side of your fingertip and press the yellow release button to prick your finger.

6

Touch the **yellow edge** of the test strip to the blood drop. Do not put blood on top of the test strip.

7

Remove your finger from the test strip when the flashing appears.

The test result appears on the display in less than 4 seconds.

Before you finish...

Remove the lancing device cap. Point the lancet away from you. Slide out the ejector to release the lancet.

LAST UPDATE: 2021-06

Roche Diabetes Care GmbH
Sandhofer Strasse 116
68305 Mannheim, Germany
www.accu-chek.com

ACCUCHEK, ACCUCHEK GUIDE, ACCUCHEK GUIDE ME, and SOFTCLIX are trademarks of Roche.

© 2021 Roche Diabetes Care
08943109061A_08943109061(01)-0921

This file may not print or view at 100%.

Die lines and color breaks do not print.

Roche USA – 83545
V4/1 – 08943109061A_08943109061(01) – Black

Roche USA – 83545
V4/2 – 08943109061A_08943109061(01) – Cyan

Roche USA – 83545
V4/3 – 08943109061A_08943109061(01) – Magenta

Roche USA – 83545
V4/4 – 08943109061A_08943109061(01) – Yellow

PMS-QUICK START GUIDE

Approved according to SHR_SOP 04.07.08_Alt A **Category 2**

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